CRYOGLUE Tissue embedding medium for cryosections

Slee **CryoGlue** is used to quickly embed fresh tissue specimens for preparation and processing of frozen sectioning using a cryostat. It is a formulation of clear, water-soluble glycols and resins. In the cryostat **CryoGlue** solidifies and evenly encapsulates the tissue specimens, bonding them to the specimen holder for consistent sectioning in a working temperature of -10 °C and below.

Dispense tissue embedding medium onto the object holder, place and orientate the tissue. Apply additional embedding medium to surround and encase the tissue specimen. Freeze the tissue to the appropriate temperature to guarantee optimum section results.

Ready to use solution. For laboratory use. Store at room temperature (15 - 25 °C) in a dry, well ventilated area. Keep container tightly closed. Dispose of the empty container according to the applicable local / governmental guidelines.

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Did you know?

CryoGlue stands for perfect cutting results. Its high viscosity results in fast freezing for optimal section quality in preparation and processing of frozen sectioning.

Choice of section thickness is important as this will have considerable effect on the final resolution at the microscope. Generally, the thickest usable sections are 10 µm and should not exceed 20 µm, since extremely thick sections will frequently shatter and the specimen becomes useless. Reduce section thickness until sectioning is just about possible (5 µm). This will greatly improve the quality.

If sections curl up or shred, try to adjust the anti-roll plate position. Minor adjustments make a big difference in tissue section quality.

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slee
solutions
for pathology

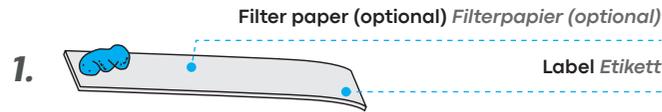
CRYOGLUE

TISSUE EMBEDDING MEDIUM
FOR SAMPLE PREPARATION
IN CRYOSECTIONING

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
DESIGN &
MANUFACTURING
MADE IN GERMANY

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Place the filter paper strip with tissue specimens on cutting chucks in the cryostat chamber and cover it with a thin layer of CryoGlue embedding medium.

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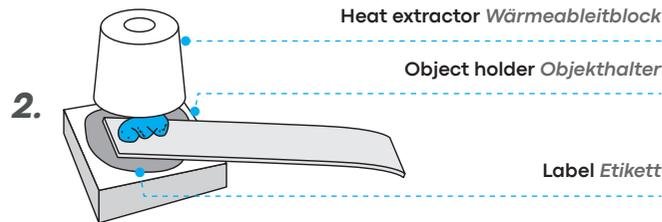Cool the specimen / chuck mount within the cryostat with a precooled heat extractor until the CryoGlue embedding medium solidifies. The specimen block must remain frozen throughout the mounting procedure to avoid formation of cryo artifacts.

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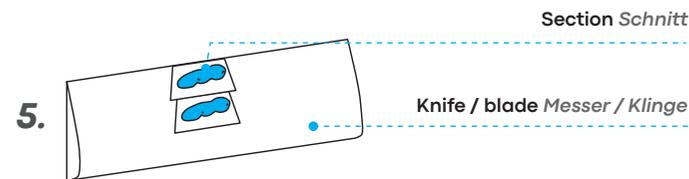
Leave specimen within the cryostat for > 10 min to cool thoroughly down to an appropriate temperature.

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Trim specimen block to a trapezoid shape using a precooled razor blade. Mind the outer boundaries of the specimen. If applicable tear off filter paper.

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Section specimen block:

- Mount the chuck in the microtome. The parallel faces of the trapezoid should be in line with the knife. Keep the wide edge toward the knife edge.
- Retract the specimen block until it easily clears the knife edge.
- Produce a smooth "face" on the block using the knife at rapid cutting speed (one section / sec).

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SLEE medical GmbH · Am Neuberg 14