

## Newsletter No. 01 / 2025

Date: Jan. 22, 2025

### **CUT 5062 +: CUT 5062 with additional coarse feed handwheel**

#### *Dear Business Partner!*

The new Slee semi-automatic microtome **CUT 5062 +** comes with all qualities of **CUT 5062**, featuring an additional coarse feed handwheel. Adding up to the motorized advance via touch panel, manual advance can be performed with the small coarse feed handwheel on the left side of the device. This combines the accuracy of motorized specimen feed with the high working speed known from manual microtomy.


#### *Key advantages:*

- Smooth and gentle horizontal object movement towards the knife or blade (decoupled, no mechanical resistance)
- Enhanced haptics
- Fast manual advance of the specimen, therefore higher working speed
- Reverse function (movement towards blade / from blade) for highest convenience

#### *How to use:*

- **Make your choice:** Both handwheel and control panel allow the motorized coarse advance.
- **There is no switching:** Handwheel and control panel can be operated permanently, at any time.
- **Easy and simple handling:** The direction of rotation can be reversed by pressing the FWD and REV buttons simultaneously while turning the device on.

#### **Impressions of CUT 5062 +:**

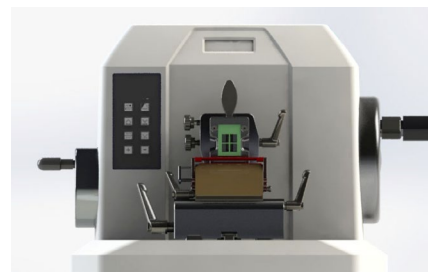

BY the way, check out our devices online, each with download area for your convenience:

*Solutions for pathology*  
**Product groups**

Slee offers a comprehensive product range for setting up histopathology labs.



**Tissue processors**



**Embedding centers**



**Microtomes**



**Cryostats**



**Water baths & slide warmers**




**Staining systems**



**Coverslippers**



**Centrifuges**

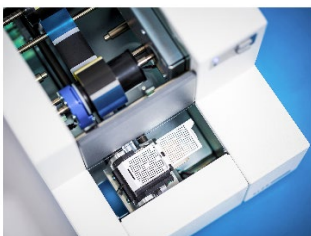

**Printers**



**Ergonomic workplaces**



**Consumables**

[Click for direct link to product groups.](#)

Questions about the products? Comments on our portfolio? Need more information? Please contact us, we will be happy to help!

Best regards,

Stefan Schock

Maged Benjamin, Simon Owen, Alexander Benjamin & Heiko Drzonek

SLEE medical GmbH • Am Neuberg 14

55268 Nieder-Olm • Germany

[www.slee.de](http://www.slee.de)

T: +49 (0) 6136 76997-0

E: [marketing@slee.de](mailto:marketing@slee.de)